

# Rapid Test Guide

# HSV 1 & 2


Type: Herpes Simplex Virus 1 & 2 Antibody IgG/IgM Test  
 Product: Rapid Test Kit  
 Specimen: Whole Blood



**Clean your finger.** Use the alcohol pad to wipe the tip of your finger.




**Prick your finger.** Use the lancet to prick your fingertip and gently squeeze until a drop of blood appears.

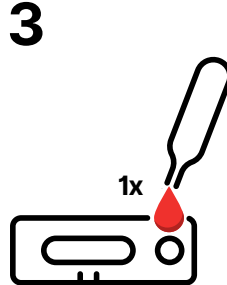


**Add blood to the cassette.** Put 1 drop of blood into the small round hole marked "S" on the test cassette.

**Note:** Make sure you add enough blood for the test to work. If the test strip doesn't start to get wet within 1 minute, add one more drop of buffer to the same spot.

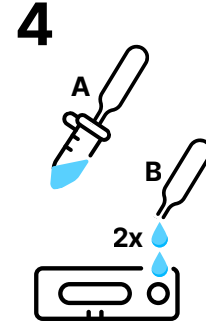


**To add buffer liquid:**

- Use pipette to get buffer from tube.
- Drop 2 drops into the round test well where blood was placed.
- Tube has extra, no need to use it all.

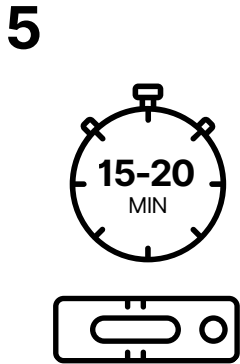

**Wait for 15 minutes.** Do not read the result after 20 minutes.

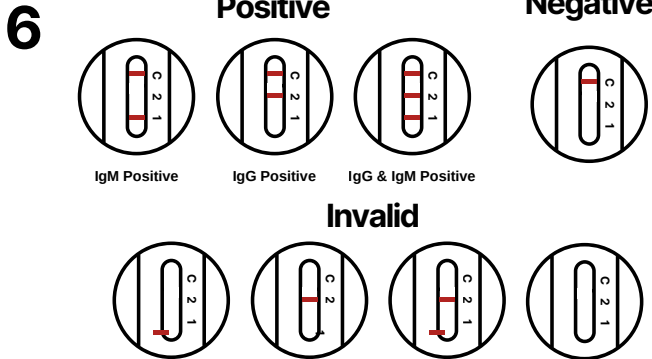

**Check the result:** IgM Positive = 2 lines (C and 1). IgG Positive = 2 lines (C and 2). IgM & IgG Positive = 3 lines (C, 1 and 2). 1 line at C = Negative. No line at C = Invalid, test again.  
**Note:** Test line intensity may be faint or dark. Any visible line at 1 or 2 should be read as positive.



**Throw away the used kit properly.** Put all used parts in a covered trash bin.