


# Rapid Test Guide

# CT/NG

Type: Chlamydia Trachomatis / Neisseria Gonorrhoeae Ag Test  
 Product: Rapid Test Kit  
 Specimen: Male Urethral Swab/Female Cervical Swab

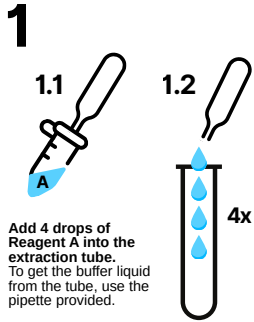


**Add 4 drops of Reagent A into the extraction tube.**  
 To get the buffer liquid from the tube, use the pipette provided.

**2**

Use the swab to collect the sample.

For individuals with a penis:

- Gently insert the swab about 1 to 1.5 inches, about (2-4 cm) into the urethra.
- Twist the swab in a full circle (360°) for 10 seconds.

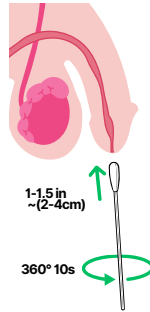

For individuals with a cervix:

- Gently insert the swab into the cervix.
- Twist the swab in a full circle (360°) for 15 seconds.

