


Rapid Test Guide

CT/NG

Type: Chlamydia Trachomatis / Neisseria Gonorrhoeae Ag Test
 Product: Rapid Test Kit
 Specimen: Male Urethral Swab/Female Cervical Swab

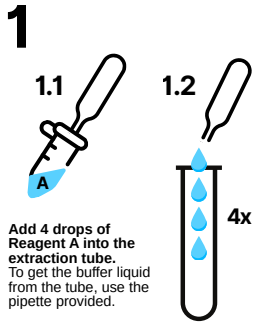


Add 4 drops of Reagent A into the extraction tube.
 To get the buffer liquid from the tube, use the pipette provided.

2

Use the swab to collect the sample.










For individuals with a penis:

- Gently insert the swab about 1 to 1.5 inches, about (2-4 cm) into the urethra.
- Twist the swab in a full circle (360°) for 10 seconds.

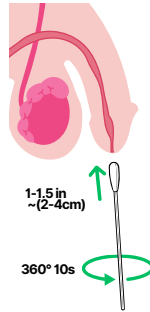

For individuals with a cervix:

- Gently insert the swab into the cervix.
- Twist the swab in a full circle (360°) for 15 seconds.

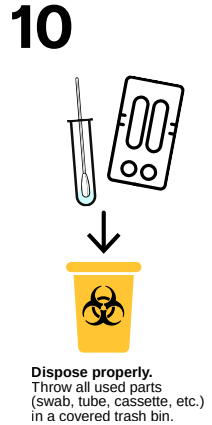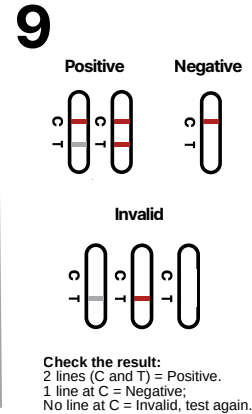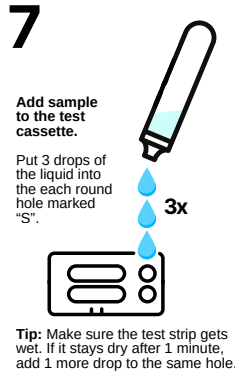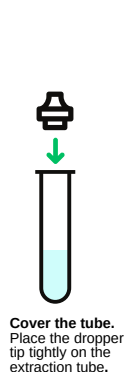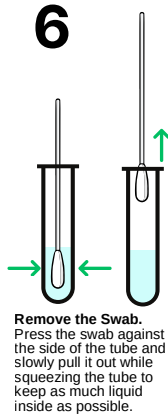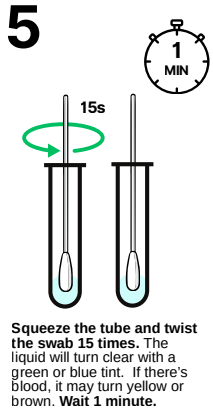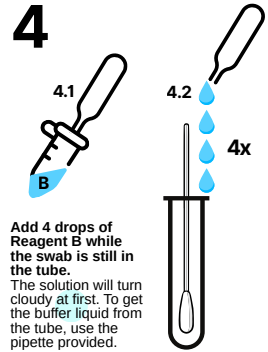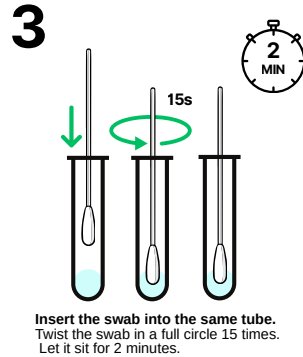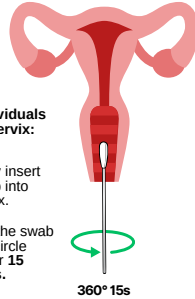

Rapid Test Guide

HIV + HBsAg + HCG + Syp

Type: 4-in-1 HIV + Hepa B + Hepa C + Syphilis

Product: Rapid Test Kit

Specimen: Whole Blood/Serum/Plasma

1




Clean your finger. Use the alcohol pad to wipe the tip of your finger.

2



Prick your finger. Use the lancet to prick your fingertip and gently squeeze until a drop of blood appears.


3

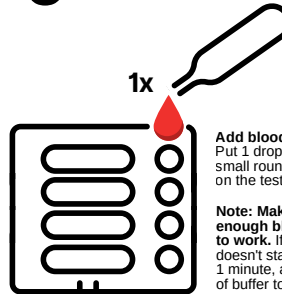

Add blood to the cassette. Put 1 drop of blood into the small round hole marked "S" on the test cassette.

Note: Make sure you add enough blood for the test to work. If the test strip doesn't start to get wet within 1 minute, add one more drop of buffer to the same spot.

4

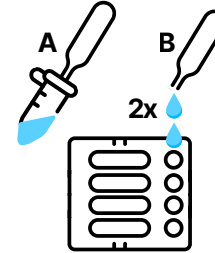

Add the buffer liquid. This cassette has 4 tests. Each one needs 2 drops of buffer.

A. To get the buffer liquid from the tube, use the pipette provided.

B. For each round hole where you placed blood:

1. Squeeze the pipette to draw buffer from the tube.
2. Gently squeeze 2 drops into each test well.
3. Repeat for all 4 wells.
4. Just 2 drops per test. The tube contains extra, no need to use it all.


5




Wait for 15 minutes. Do not read the result after 20 minutes.

6


Positive

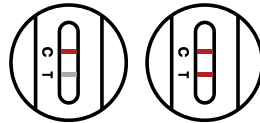

Negative

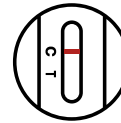

Invalid

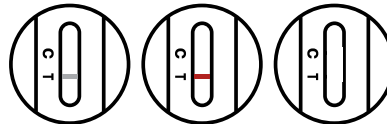

Check the result: 2 lines (C and T) = Positive. 1 line at C = Negative. No line at C = Invalid, test again.

7



Throw away the used kit properly. Put all used parts in a covered trash bin.